INFORMATION FOR PATIENTS

MuGard[®] Oral Mucoadhesive provides effective management of oral mucositis

How does *MuGard®* work?

When you use *MuGard*, it helps you manage your symptoms of oral mucositis. *MuGard* coats the lining of your mouth with a soothing, protective layer that helps manage the mouth and throat soreness caused by oral mucositis.

MuGard has been tested in a clinical trial. In the trial, *MuGard* reduced the mouth and throat soreness of oral mucositis in patients who were being treated with chemotherapy and radiation therapy.

Indication: MuGard[®] Oral Mucoadhesive is indicated for the management of oral mucositis/stomatitis (that may be caused by radiotherapy and/or chemotherapy) and all types of oral wounds (mouth sores and injuries), including aphthous ulcers/canker sores and traumatic ulcers, such as those caused by oral surgery or ill-fitting dentures or braces.

Contraindications: *MuGard* is contraindicated in patients with known hypersensitivity to any of the ingredients in the formulation.

Please see Important Safety Information on page 6 and accompanying full Prescribing Information, also available at MuGard.com.



1





How do I get MuGard® Oral Mucoadhesive?

MuGard is available by prescription only. You can obtain *MuGard* in either of the following ways:

- AMAG Assist[®] will work to provide *MuGard* directly to your home by mail order or through a specialty pharmacy.
- You can pick up *MuGard* at your local pharmacy.

A Patient Assistance Program may be available if you don't have commercial insurance coverage. It may also be available if your insurance does not cover *MuGard*.*

• MuGard may be provided at no charge to patients who meet financial and other criteria.

For more information, call AMAG Assist at 844-635-AMAG (2624)

* Restrictions apply. Patients must be at least 18 years of age. They must be residents of the United States. They may not have any private or public health insurance coverage for *MuGard*. They also must meet income requirements. Medicare patients with no supplemental insurance coverage for *MuGard* may be eligible.

When should I get my *MuGard*®?

Talk to your doctor to obtain a prescription for *MuGard* before your cancer therapy begins. You can start using *MuGard* as early as the first day of therapy and throughout your course of treatment, or as your doctor directs.

Are there any side effects with *MuGard*®?

In a clinical trial, no significant side effects were reported in patients taking *MuGard*. Also, when *MuGard* is used with other medicines, there are no known interactions.

Please see Important Safety Information on page 6 and accompanying full Prescribing Information, also available at MuGard.com.





How do I use MuGard® Oral Mucoadhesive?

MuGard comes ready to use-no mixing or diluting is recommended or required.

- Gently pour 5 mL of *MuGard* in your mouth. Use the measuring cup included. Be sure to coat your entire mouth with *MuGard*.
 - \pm You can use up to 10 mL if needed to coat the inside of your mouth fully.
- Swish *MuGard* in your mouth for a minute or longer, if you can.
 - You may find that swishing is too difficult or painful. If so, try spreading *MuGard* throughout your mouth by gently rotating your head.
- After coating, you can either spit out *MuGard* or swallow it.
- Use MuGard 4 to 6 times daily.
- After using MuGard, wait at least 1 hour before eating or drinking.

What else should I know when using *MuGard*®?

Be sure to follow these precautions when using MuGard:

- Avoid eating or drinking for at least 1 hour after using *MuGard*.
- After using *MuGard*, replace the bottle cap and tightly seal the bottle.
- Do not use MuGard after the expiration date shown on the product label.
- Do not use *MuGard* if you are allergic or sensitive to any of its ingredients.
- Use MuGard as directed. Do not dilute or mix MuGard with water.

Please see Important Safety Information on page 6 and accompanying full Prescribing Information, also available at MuGard.com.



Indication and Important Safety Information

Indication: MuGard[®] Oral Mucoadhesive is indicated for the management of oral mucositis/stomatitis (that may be caused by radiotherapy and/or chemotherapy) and all types of oral wounds (mouth sores and injuries), including aphthous ulcers/canker sores and traumatic ulcers, such as those caused by oral surgery or ill-fitting dentures or braces.

Contraindications: *MuGard* is contraindicated in patients with known hypersensitivity to any of the ingredients in the formulation.

Special Precautions for Use: Avoid eating or drinking for at least one hour after rinsing with *MuGard*. After use, replace the bottle cap and tightly seal the bottle. Do not use this product after the expiration date shown on the product label. Do not use *MuGard* if you are allergic to any of its ingredients. Dilution of the product prior to use is not recommended.

Please see accompanying full Prescribing Information, also available at MuGard.com.



AMAG Pharmaceuticals, Inc. | 1100 Winter Street | Waltham, MA 02451

AMAG Pharmaceuticals and the AMAG logo are registered trademarks of AMAG Pharmaceuticals, Inc. and the MuGard and MuGard logo are registered trademarks of Abeona Therapeutics Inc. ©2017 AMAG Pharmaceuticals, Inc. PP-MGD-US-00007

